



INCONTINENT COVERS

Our incontinent covers are: fluid-proof, which means that the fabric is fluid proof, however, once manufactured it is possible for small amounts of fluid to enter through needle holes in the fabric. Covers are manufactured to reduce this possibility, and are also: non-allergenic, anti-bacterial, anti-microbial and help reduce both shear and friction for increased patient comfort.

CLEANING INSTRUCTIONS

ROUTINE CARE – Wipe clean with neutral soap suds and lukewarm water. Rinse and dry.

HARD TO CLEAN AREAS – Soft sponge with liquid cleaner as specified on manufacturer's product label. **DO NOT USE HARSH CLEANERS OR SOLVENTS.**

NOTE: Laundering is NOT recommended.

Disinfection – Most phenolic or quaternary type disinfectants can be used. User must dilute disinfectants and germicides in accordance with manufacturer's instructions. It is recommended that all cleaning agents first be spot tested on an inconspicuous part of the fabric to ensure that they do not damage the fabric. If hydrogen peroxide based cleaners are being used they must have a Ph reading of 7.0 or above. Anything below 7.0 will cause fabric damage.

Blood Contamination – Contamination with blood can be disinfected with a 1:10 dilution of household bleach (5.25% sodium hypochlorite) as recommended by the CDC (US Dept. of Health and Human Services, February 1989).

Odor Control – To prevent odors on long term incontinent applications, clean daily. The use of a scented cleaner/disinfectant is recommended.

WARNING:

- Iodophor type disinfectants (Betadine, for example) will stain fabric.
- Prolonged contact with bleach solutions, improper dilution ratios for any cleaning agent, or poor rinsing of fabric surface after disinfecting and cleaning may cause damage to both the appearance and performance of this fabric.
- Surface Mildew – Wash with diluted bleach, use a soft brush for stubborn growth. Rinse repeatedly with clean, cold water.
- Urine – Sponge with soapy water containing a small amount of household ammonia. Rinse thoroughly with clean water.

FABRIC COVERS

QUILTED ORGANIC COTTON AND CERTIFIED ORGANIC COTTON COVERS

The Quilted Organic and Certified Organic cotton fabrics are offered as an alternative to the incontinent cover. These stretch-knit fabrics are light ivory in color, with an imprinted pattern. Please note that this material is NOT INCONTINENT, and should not be used in situations where the client suffers from incontinence or hyperhidrosis.

While it is not recommended to remove the cover, should it become necessary, the cleaning instructions are as follows:

CLEANING INSTRUCTIONS

Washing: Cold Water Wash – DO NOT USE BLEACH
Drying: Hang to dry. DO NOT use a machine dryer

Dry Cleaning: (Perchlorethylene) - is acceptable

PERCALE (POLY/COTTON) COVERS

The Percale cover is a non-stretch, ivory colored poly/cotton blend. Please note that this material is NOT INCONTINENT, and should not be used in situations where the client suffers from incontinence or hyperhidrosis.

While it is not recommended to remove the cover, should it become necessary, the cleaning instructions are as follows:

CLEANING INSTRUCTIONS

Washing: Cold Water Wash – DO NOT USE BLEACH
Drying: Hang to dry. DO NOT use a machine dryer