

**CONFORM® Products are Medical Product Class I**  
(Reg. Nr. DE/CA/23/051603NA001.01.)



**CE Declaration:**  
CONFORM® Air Cushion Systems are in accordance with all requirements for medical products as set out in 93/42 EWG, made in the EC.

**Manufacturer:**  
WOLFF KUNSTSTOFFE GmbH  
Industriestraße 28  
69509 Mörlenbach / Germany

**Fiscal No.:** 007 248 13204  
**VAT No.:** DE 1444586680

**Certified according to:**  
DIN EN ISO 9001:2008 and  
DIN EN ISO 13485:2010



**CONFORM®**  
Medical Air Cushion Systems

A brand of  
Wolff Kunststoffe GmbH  
Industriestraße 28  
69509 Mörlenbach - Germany  
Tel. +49(0) 6209 / 7200-34  
Mobil +49(0) 176 / 20 36 78 24  
Fax +49(0) 6209 / 7200-20  
a.roedel@wolff-kunststoffe.de  
www.conform-sitzkissen.de

Andrea Rödel  
Managing Director  
CONFORM®  
Sales & Marketing

# CONFORM®

## OPERATING INSTRUCTIONS MATTRESS OVERLAY SYSTEM

48"  
19" x



# CONFORM®

# CONFORM®

**Depending, what kind of version – delivery contains at:**

**BMC3:**

3 CONFORM® inflatable mattress sections, 1 foam section (foam equipped with waterproof cover), connector stripes and pump

**BMC4:**

4 CONFORM® inflatable mattress sections, connector stripes and pump


**BMC2:**

2 CONFORM® inflatable mattress sections, 2 foam sections (foam equipped with waterproof cover), connector stripes and pump

**Assembly of the mattress**

Assembly of the mattress is best carried out on top of the bed. Lay out the inflatable mattress sections parallel to each other on top of the bed.


Connect the mattress sections with the enclosed connector strips. The connector strips are attached by pulling the air cells of the mattress segments through the prepunched holes of the connector strip. Continue connecting all the mattress segments in this way.

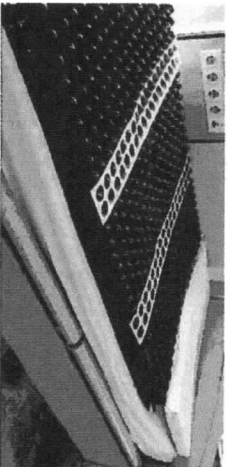

The head section of the mattress has an integrated connector strip attached to the waterproof covering, connect this foam section to the inflatable mattress section as described previously. The mattress is now complete.

**Inflation of the mattress and remark**

Open one of the two valves on each of the mattress sections and inflate each section with the enclosed pump until it begins to arch. Cover the entire mattress with a loose bedsheet or our waterproof cover (please consider that natural rubber should not be in contact with cremes).

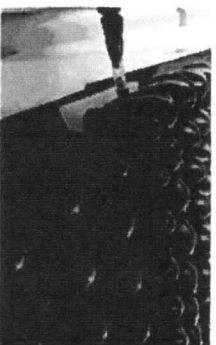

**Positioning**

Transfer the patient onto the mattress and adjust the system to the patient in the following manner:

Position the patient so he is lying on his side, the trochanter being deepest point of contact with the mattress system. Position your hand below the patients trochanter/hip (between the bedcover and the CONFORM mattress system) and check the distance to the base of the CONFORM section. The distance between the patients skin and the base of the CONFORM section should be between 2 - 2,5 cm.

Adjust each of the mattress segments in this manner starting at the trochanter, shoulders and finally the heels.

It is sufficient to check the mattress from time to time to assure it has air if so, please inflate again some air. Ensure that the valves are closed time.



**Cleaning**

For disinfection we recommend common disinfection spray used in hospital care. For cleaning the mattress sections, tear off the connector strips. Now the single sections can be moved from the bed easily for cleaning. The section can be cleaned in the washing machine at max. 40° (make sure the valves are closed!) or it can be cleaned in disinfection spray or even with a pressure cleaner. Please do not dry the inflatable sections under UV light or in the sun.