

CAUTION!

To avoid premature degradation of the surface coating never use an abrasive cloth, sponge or other similar device on the surface.

Carefully follow manufacturer instructions for recommended exposure time of cleaner on surface. Wipe off cleaning product residue with a clean, soft cloth after cleaning and allow the surface to dry completely before use.

IMPORTANT INFORMATION:

Cleaners/disinfectants can cause discoloration/bleaching on some colors. To avoid discoloration, always check on a small area first.

Information contained here is purely related to the compatibility of cleaner materials with EMP grey Vintex fabric and in no way relates to the efficacy of any cleaner for infection control or the safety of any cleaner for skin contact.

Antibacterial / Antistatic Wipedown Flame Retardant Fabric

Care and Maintenance

Soil/Stains: Use mild soap suds and lukewarm water, rinse well with water and allow to dry. **DO NOT USE HARSH CLEANSERS, SOLVENTS OR DETERGENTS.**

Hard to Clean Spots: Use standard vinyl cleansers and soft bristle brush on troublesome spots or stains.

Disinfection: Dilute disinfectants and/or germicides as specified on the product label.

DO NOT USE CONCENTRATION THAT EXCEEDS THE MANUFACTURER'S RECOMMENDATIONS.

Infectious Patients: Many healthcare facilities have been advised to use the following disinfectant, which can be used on surfaces where EMP Grey Vintex fabric is used.

- **0.525% Sodium Hypochlorite solution. You can achieve this by diluting 1 part Clorox® brand bleach with 9 parts water.**